

Updating Process of Zircom software

SD card is necessary for updating. Please prepare a SD card beforehand.

※ Firing schedule is not changed by this updating.

◇Procedure

- ① Decompress the ZIP file downloaded from our company HP, then copy the following data to the route folder of SD card.
 - ZRM_Vxx.mot (program file xx : Program Version No.)
 - ZRM1_Eyy (display-data-folder yy : Display Version No.)
- ② Insert SD card(①) in SD slot, and turn on a power supply.
- ③ Update according to a display.

(1) Please press START/STOP key. Updating starts.

ZRM_Vxx.MOT : 20YY/MM/DD hh:mm:ss

[Program] updating ?

Yes : [START/STOP] KEY


No : [OTHER] KEY

(2) Updating is completed in several seconds. Please press the START/STOP key.

Update OK

Please [START/STOP] key

(3) Update display-data. Please press the START/STOP key. It takes several minutes.
Update will be completion if "the update OK" is displayed on the last line.

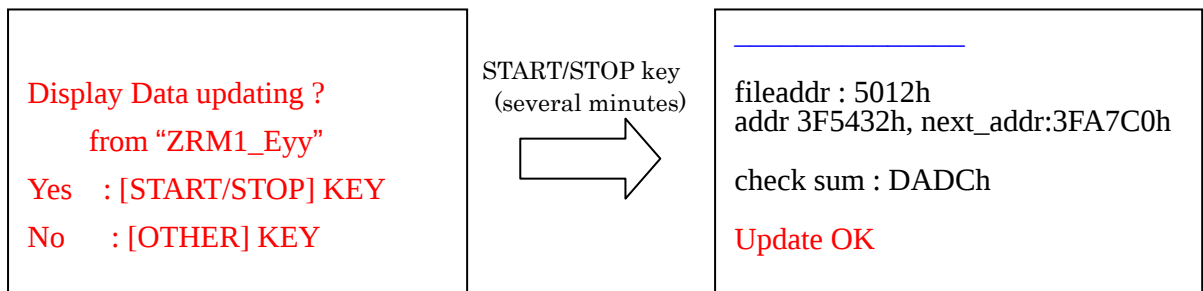

The updating is finished. Please remove the SD card.